

# It's Time To See Your Future



## My Valeda Treatment

**You are being offered treatment with the Valeda Light Delivery System because you have been diagnosed with dry age-related macular degeneration (AMD).**

### Dry Age-Related Macular Degeneration

AMD is a progressive disease of the retina, the part of the eye that detects visual signals and sends them to the brain. It is the leading cause of vision loss in people over 65, and there is no cure.

There are two forms of AMD:

- The dry form accounts for 85 to 90% of AMD cases
- The wet form accounts for 10 to 15% of AMD cases

### Benefits of Valeda Treatment<sup>1</sup>

Treatment with Valeda can help you see more clearly now and in the future. Retinal imaging indicates improvement in eye scans when assessed for drusen (protein deposits observed in eyes with dry AMD).

### How Valeda Works

Valeda uses photobiomodulation (PBM), specific wavelengths of low-level light therapy, to stimulate energy production within the eye resulting in improved cell health. Healthy retinal cells are important for good vision.

## Preparing for Your Valeda Treatment

No special preparation is needed before treatment. You will need to remove glasses or contact lenses. There are no eye drops, drugs, or needles used in the treatment. There is no discomfort during or after the treatment.

## During Your Valeda Treatment

You will be seated comfortably in front of Valeda and will be guided through each step of the treatment. The active treatment time is less than 5 minutes per eye per session.

You will receive 9 treatments over 3 to 5 weeks during each series. Typically, a series is repeated every 4 months. Your doctor will determine your plan.

## After Your Valeda Treatment

Due to the exposure to bright lights, you may experience an afterimage following treatment. This is known as photostress. Normal photostress recovery time is approximately 15-30 seconds. Following treatment, you will be able to resume normal activities.

## Valeda Safety

There have been no treatment-related side effects in clinical trials with Valeda.

Consult your eye doctor for an explanation of the benefits and risks associated with Valeda treatment.

Valeda is CE Marked and authorized by the U.S. Food & Drug Administration (FDA).

---

## Valeda Indications for Use

The Valeda Light delivery system is indicated use for the treatment of ocular damage and disease using photobiomodulation, including inhibition of inflammatory mediators, edema or drusen deposition, improvement of wound healing following ocular trauma or surgery, and increase in visual acuity and contrast sensitivity in patients with degenerative diseases such as dry age-related macular degeneration.

Tel: +44 (0) 1270 567477 | Email: [lianne.stobo@eyecorrectioncentre.co.uk](mailto:lianne.stobo@eyecorrectioncentre.co.uk)



EYE  
CORRECTION  
CENTRE



[www.eyecorrectioncentre.co.uk](http://www.eyecorrectioncentre.co.uk)

MKT-0135 enGB REV A  
© 2025 LumiThera, Inc.